


# Holter ECG Setup

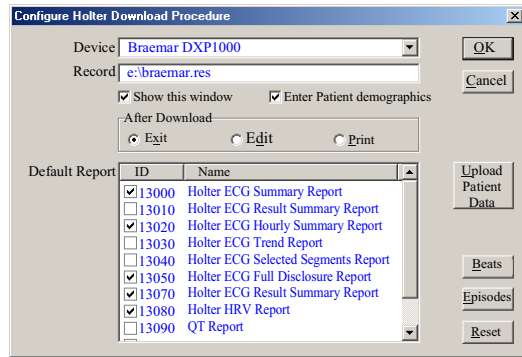

## Upload Patient Data:

Type in Patient Data before Holter Hookup  
(This option must be supported by the Recorder)

## Beat:

Advance Users - See User Guide

## Episodes:

Advance Users - See User Guide

## Device:

Type of Recorder Used  
(Braemar DXP, Datrix etc)

## Record:

Recorder Drive/File Location  
Recorder/File location e.g d:, e: etc

## Show this window:

Shows this window when "HOLTER" button is pressed

## Enter Patient Demographic:

Enter patient information during holter data download from recorder  
(In some recorder patient data can be entered patient ecg is recorded)

## After Download:

### Exit

After holter data download program goes back to Cardiology Suite Master Screen

### Edit

After holter download program goes directly into Edit Menus

### Print

After holter download generates directly auto reports (No edit)

## Holter Process

- Hook up patient and record 24hr Holter ECG
- Download 24hr ECG to computer / Auto Analysis  
Connect Holter Recorder to USB Port of your computer  
Run QRS-Card Cardiology Suite Software  
Press "Holter" Button on the Tool Bar  
Press "Holter" Tab to check the correct recorder selection  
Press "Patient" Tab to type in Patient Demographics  
Press "OK" Button to Start Download / Auto Analyze Holter data  
Press "Open" Button on the Tool Bar to load the analyzed data
- Review / Edit Holter ECG data Manually
- Select Strips / Episodes to Print
- Type in Physician comments
- Print Final Holter Reports

# Holter ECG Edit Screens

