


Print ECG  
If not Auto Print  
Single Style Report

Finish Test  
If not Auto Save

ECG Measurements

Heart Rate

Blood Pressure

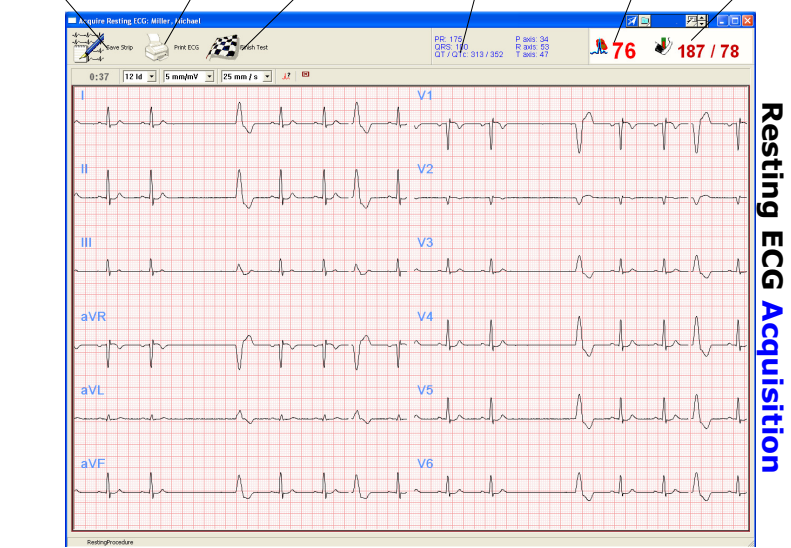

Resting ECG Acquisition

**Resting ECG Acquisition**

# QRS-Card Cardiology Suite 4.0

## Resting ECG, Stress ECG, Holter ECG & ABPM

Search by Options

Open Patient

Print Report

Add New Patient

Delete Patient

Patient Database

Test Report Preview

**MAIN SOFTWARE SCREEN**

**About QRSCard Cardiology Suite 4.05**  
 QRS-Card Cardiology Suite 4.05 is collection of Cardiology Software with single unique user interface and a combined database for Resting ECG, Stress ECG, Holter ECG and Ambulatory BP.

QRS-Card Cardiology Suite 4.05 software requires a USB Security Key for performing a real patient test. A demo driver is provided for Resting, Stress and Holter software for tutorial and Software testing purposes. QRS DEMO DRIVER can be selected (full software functionality) under Resting, Stress or Holter Setup under "Recorder" (CTL+F2, CTL+F3, CTL+F3)

The user must first setup and register the software.

To Setup:  
 Click File Resting, Setup or CTL+F2

Registration of the software can be performed:  
 a. From Master Screen Press **CTL+F1**  
 b. To Register software License Press **ALT+R**  
 (USB Security Key & a 17 Digit PID code is required for registration/software activation)

DEMO Program / DRIVER does NOT require a PID

PID (Product ID)

USB Key SN (Serial Number)

Patient Data

ECG Data


Tabular Data

Reports

Help

Heart Rate

Blood Pressure

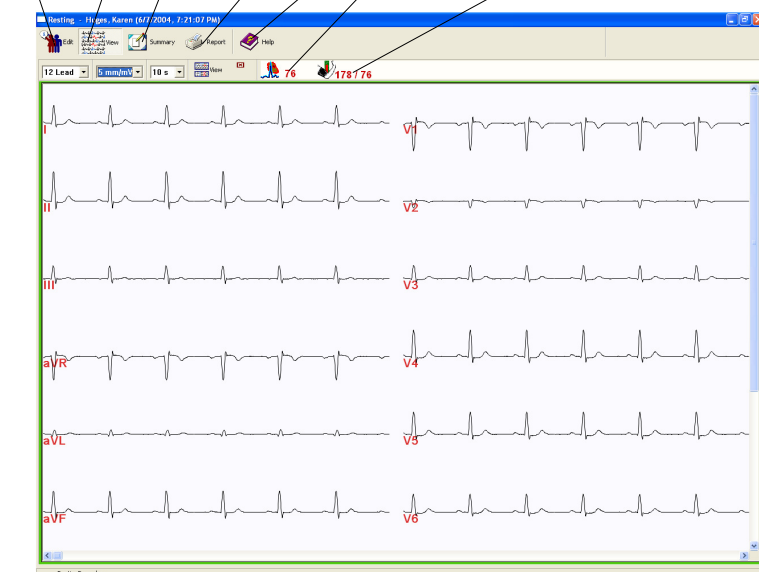

Resting ECG DataView

**Resting ECG DataView**



**Pulse Biomedical, Inc.**  
 1305 Catfish Lane, Norristown, PA 19403, USA  
 Tel: 610-666-5510 Fax: 610-666-5630

**Pulse Biomedical GmbH**  
 Wilhelm Bihler Str 4, 72474 Winterlingen, GERMANY  
 Tel: +49-7434-316038 Fax: +49-7434-316039



**Pulse Biomedical, Inc.**  
 1305 Catfish Lane, Norristown, PA 19403, USA  
 Tel: 610-666-5510 Fax: 610-666-5630

**Pulse Biomedical GmbH**  
 Wilhelm Bihler Str 4, 72474 Winterlingen, GERMANY  
 Tel: +49-7434-316038 Fax: +49-7434-316039