

Resting Setup Window

ID	Name
<input type="checkbox"/>	1200 12 Lead Standard
<input type="checkbox"/>	1201 12 Lead Standard (Notes)
<input checked="" type="checkbox"/>	1202 12 Lead Standard (Detail)
<input type="checkbox"/>	1203 3 Lead Standard
<input type="checkbox"/>	1204 3 Lead Standard (Notes)
<input type="checkbox"/>	1205 3 Lead Standard (Detail)
<input type="checkbox"/>	1206 6x2 12 Lead Standard

Resting Procedure Tasks

- Show this Window-** Shows setup window each time 'RESTING' button is selected
- Show Electrode Position Diagram-** Shows electrode positioning diagram before start of Resting ECG test
- Collect blood pressure measurement-** Allows user to input blood pressure before Resting ECG acquisition
- Start test when measurement valid-** Stores Resting ECG only when good/valid ECG acquisition
- Provide automatic ECG Interpretation-** Performs automatic ECG Interpretation before storing ECG
- Print ECG Report-** Print Resting ECG report automatically after test completion
- View test record-** Allows user to view Resting ECG data after test completion
- Exit when all task complete-** Closes Cardiology Suite Master screen after test completion

ECG Filter

- Electrical Interference:** 60Hz or 50Hz
- Artifact rejection:** None or Standard
- Baseline Drift Reduction:** On or None

You can also select **DEMO** to preview selected settings or run a demo of the software.

Default Reports

Select the reports required

ECG Acquisition

- Recorder:** Select device
- Sampling rate:** 240
- Length:** Total Resting ECG Test time in seconds
- Average Cycle:** No of beats for Averaging complex

Easy Steps To Perform Resting ECG

- Press Resting Button to Start a Test
- Enter Patient Demographic data
- Auto or Manual Print Test Report

